Product name
CINOSCAR silicone sheets - washable

Product description
- CINOSCAR silicone sheets are a proven technique for treating keloids and hypertrophic scars and preventing abnormal scar development after surgery or injury.
- The silicone sheet is self-adhesive, washable and can be used for long periods of time. There are several variants – a transparent silicone sheet and variants offering UV protection, which come in various thicknesses.

Composition of silicone sheet with UV protection
- Textile layer: polyamide, elastane
- Medical-grade silicones

Composition of transparent silicone sheet
- Polyurethane film
- Medical-grade silicones

Propertes
- CINOSCAR silicone sheets make scars flatter, smoother and thinner and reduce complaints such as persistent redness, pulling of the scar, itching, restriction of joint movement, discomfort and cosmetic concerns.
- The silicone sheet provides a semi-occlusive (covering) layer on top of the skin, which results in increased hydration of the skin (epidermis). This increased hydration ensures that the damaged, scarred skin recovers more quickly, and that functional concerns (complaints) decrease.
- CINOSCAR silicone sheets with UV protection blocks at least 93% of UV radiation.

Indications/applications
CINOSCAR silicone sheets are suitable for:
- both old and new scars
- all skin types
- all ages
- the treatment and prevention of abnormal scars (hypertrophic scars and keloids)
- the treatment of surgical scars
- use under clothing and compression therapy
- the treatment of persistent functional concerns related to the scar (redness, pulling of the scar, itching, restriction of joint movement, discomfort and cosmetic concerns).

Instructions for use
- For optimal results, preferably start CINOSCAR silicone therapy immediately after wound closure (after sutures are removed and any scabs have disappeared).
- It is recommended to build up CINOSCAR silicone therapy gradually - to a maximum of 23 hours a day. This gradual increase allows the skin to get used to the silicone.
- For best results, it is preferable to wear the silicone sheets for a minimum of 12 to a maximum of 23 hours a day.
- If necessary, the CINOSCAR silicone sheet can be cut to the right size.
- Make sure the silicone sheet extends 2 cm beyond the edges of the scar.
- For good adhesion and durability of the silicone sheet, it is advisable to remove any hair around the scar site.
- Before applying the silicone sheet on and around the scar, clean and dry the skin thoroughly.
- Remove the protective film from the back of the sheet and stick the CINOSCAR silicone sheet onto the scar. Sometimes the protective film consists of several parts. In such case, first remove the first part of the protective film and carefully apply it at the start of the scar. Then remove the next part of the protective film and apply the next part of the silicone sheet to the scar. Depending on the size of the silicone sheet, the above should be repeated until the entire silicone sheet has been applied.
- If necessary, the silicone sheet can be fixed with an adhesive or elastic bandage.
- After removal of the silicone sheet, the skin may be treated with a hydrating cream.

Product name
CINOSCAR silicone sheets - washable

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Care instructions
- It is necessary to clean CINOSCAR silicone sheets thoroughly, on a daily basis. This is important to maintain good adhesion.
- Rub the silicone sheet with a few drops of neutral soap/lotion.
- Remove as much dirt as possible with soap and lukewarm water.
- Rinse the silicone sheet thoroughly under lukewarm running water.
- After contact with soap and water, the adhesive strength disappears and the silicone sheet may roll up.

- Dry the silicone sheet on a clean, dry surface, with the sticky side facing up.
- As the sheet dries, the adhesive strength gradually returns and the dressing will unroll itself.
- Use the silicone sheet only after it is completely dry.
- The edges of the silicone sheet can be trimmed if the adhesive strength deteriorates due to use.

How long can I use a CINOSCAR silicone sheet?
- It takes at least 3-6 months for a scar to normalise.
- Apply CINOSCAR silicone sheets as long as the scar is raised, red or causing complaints.
- Consult your therapist/doctor on the duration of therapy.

Duration of use
- CINOSCAR comes in packages of one or more silicone sheets.
- When used as maintained according to the instructions for use, the contents of a pack of CINOSCAR are sufficient for approximately 3 to 8 weeks of treatment.
- Multiple packs may therefore be necessary for long-term therapy.

Special attention and precautions
- Keep out of reach of children.
- CINOSCAR silicone sheets are only suitable for external use.
- Use CINOSCAR silicone sheets only on healed, closed skin (after the scabs have disappeared) and never on open wounds.
- Never dry the CINOSCAR silicone sheet in the sun or near a heat source.
- Remove the CINOSCAR silicone sheet while exercising or swimming. Perspiration or contact with water can cause the CINOSCAR silicone sheet to loosen.
- In general, limit exposure of scars to sunlight, as much as possible.
- Do not use if you are hypersensitive to any of the ingredients.
- Do not use if the packaging is damaged on receipt of the product.

Side effects

- In rare cases, the skin may have a hypersensitive reaction (skin irritation) to silicone treatment. In such cases, stop treatment until the skin calms down.
- Re-commence treatment and let the skin get used to the CINOSCAR silicone sheet gradually (by extending treatment by an hour each day).
- If skin problems persist, consult your therapist/doctor.
- If there are serious adverse events related to the product, these should be reported to the manufacturer and the competent authorities (of the member state) where the user is located.

Storage
- The expiry date is stated on the packaging.
- CINOSCAR silicone sheet with UV protection can be stored in unopened packaging for 3 years.
- CINOSCAR transparent silicone sheet can be stored in the unopened packaging for 5 years.
- Store at room temperature and not above 25°C.

Date of last revision of the instructions for use
May 2023

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